




The T-Safe® Flex Wall shower is a permanent and flexible shower solution, which can be used with and without filter. It retains the water free from bacteria such as Legionella pneumophila, Pseudomonas aeruginosa, Brevundimonas diminuta and Aspergillus brasiliensis (spores).

Features:

- 0.1 µm sterile-graded hollow fibre membrane filter
- High capacity filter, which secures up to 92 days lifetime without loss of comfort
- T-Safe® Flex can be installed directly on most existing installation
- Water connection from internal or external pipe (from top, bottom or side)
- Built-in stop valve and flow restrictor, which provides a long-term constant flow
- Adaptor with non-return valve available on request
- Dummy must be installed when filter is not needed as to release the stop-valve
- Smooth surfaces makes daily cleaning
- Easy Clean faceplate. Easy replaceable

Easy Clean:

The Easy Clean rubber nozzles expands depending on the water pressure. In this way limestone will flush out. More stubborn limestone can be removed by rubbing the nozzles.

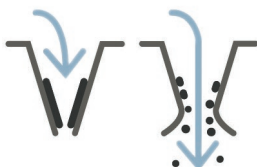


The Flex shower offers a flexible installation as water connection can be made from the behind, from the top, from the side or the bottom.

Technical Data:

Max. Operating pressure:	6 bar
Max. Operating temperature:	60°C
Mean flow rates with 8L restrictor:	4.3 L/min at 1 bar 6.3 L/min at 2 bar 7.2 L/min at 3 bar 7.7 L/min at 4 bar 8.3 L/min at 5 bar
Filter lifetime:	92 days*
Norminal use capacity:	>7500 litre
Membrane area:	5090 cm ²
Bacteria retention:	>Log7
Connection:	1/2" - internal or external pipe
Material:	Glass Reinforced Nylon
Colour:	Silver coated

*) The filter ensures bacteria-free water for up to 92 days - or until there is a blockage, whichever occurs first. If SDI>2 the capacity is reduced.

Note: The filter cannot withstand thermal disinfection

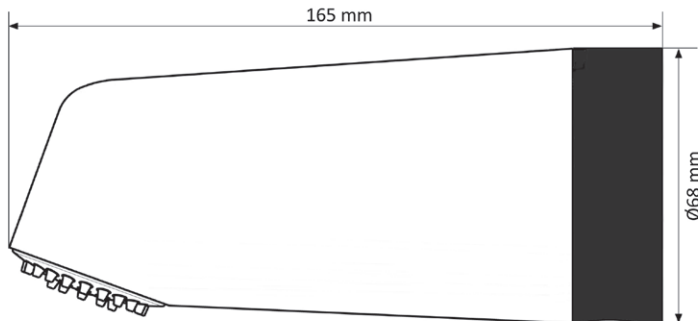

Product variants	T-Safe® Flex Wall shower
Shower with filter	02-803170
Shower with dummy	02-803171
Filter only	02-000187
Dummy only	02-000186

The T-Safe® filter has been tested in accordance to ASTM F838-05 with the following microorganisms:

- Brevundimonas diminuta
- Pseudomonas aeruginosa
- Legionella pneumophila
- Aspergillus brasiliensis

References:

- Miscellaneous external and internal tests

The information provided in this document was reviewed for accuracy at the time of publication. Product data may be subject to change without notice. For current product data and product availability in specific geographical areas please contact your local T-Safe® distributor or Tandrup Water Solutions (HQ).

ISO 9001 - ISO 14001



Tandrup Water Solutions A/S

Gydevang 1 • DK-3450 Allerød • Denmark

Phone: (+45) 48 17 22 82 • E-mail: tws@tandrup.dk • www.t-safe.com